

DISCOVER THE POWER OF CONNECTION

MINIMED 640G®





INSULIN PUMP THERAPY CHANGING LIVES TODAY



Have you just been diagnosed with insulin dependent diabetes? Perhaps you've been on multiple daily injection therapy for years? Or maybe you're caring for someone?

Whatever the situation, Medtronic is here to support you in better managing your diabetes with insulin pump therapy.

An insulin pump is a discreet electronic device, about the size of a small mobile phone. It closely mimics the way a healthy pancreas functions, replacing the need for frequent injections by delivering precise doses of rapid-acting insulin 24 hours a day to closely match your body's daily needs.

HOW DOES THE PUMP WORK?

The pump provides insulin to your body at two rates

1

BASAL RATE

Small amounts of insulin released continuously throughout the day to mimic the background insulin production of the pancreas.

2

BOLUS RATE

Additional insulin can be delivered "on demand" to match the food you are going to eat or correct high blood sugar.

COMPONENTS OF INSULIN PUMP THERAPY



1. INSULIN PUMP

A small, durable electronic device to program your insulin & show you how you are tracking. The device includes a reservoir compartment.



2. INFUSION SET

An infusion set includes a thin cannula that goes from the reservoir to the infusion site on your body. The cannula is inserted into the site - similar to where you would give insulin injections. The infusion set is changed every two to three days.



3. RESERVOIR

A plastic cartriage that holds the insulin and is locked into the insulin pump. A reservoir can hold up to 300 units of insulin and is changed every two to three days.

INSULIN PUMP THERAPY THE BENEFITS



Insulin pump therapy offers you many benefits over multiple daily injections (MDI) such as ^{1,2}:

- Better HbA1c control
- Fewer hypoglycaemic events
- Reduction in glycaemic variability.

It can also help to better manage the need for insulin dose adjustment, particularly for meals, exercise, on sick days and overnight - helping you achieve better glucose control.

Switching to an insulin pump like the MiniMed® 640G, replaces the need for multiple daily injections for better control over your glucose levels. It is easier to do things like eating out, sleeping in on the weekends, playing sports, travelling and even swimming as the pump is waterproof up to 3.6m for 24 hours (IPX8).

FEATURE	PUMP	MDI
Fewer Daily Injections	✓	X
24/7 Insulin Infusion	1	Х
High and Low Alerts & Alarms	✓	Х
Insulin Suspension to Avoid Hypos	1	Х
Bolus Calculator	✓	Х
Ongoing Insulin Tracking to Avoid Stacking	1	X
Visualise your Glucose Data	1	Х

LOWER HBA1C^{1,2}









NERVE DAMAGE REDUCED UP TO 4

THE MINIMED® 640G HOW DOES IT WORK?



PROTECTING YOUR QUALITY OF LIFE

- 93% of pump therapy users preferred their pump compared to MDI $^{\scriptscriptstyle 5}$
- 90% less injections compared to MDI³
- · Adjusts for your individual basal insulin needs


PROTECTING YOUR HEALTH OUTCOMES TODAY

- Improved HbA1c²
- Improved glycaemic variability²

PROTECTING YOUR FUTURE

 Reduction in diabetes complications such as cardiovascular disease, coronary disease and total mortality⁶.

HOW SMARTGUARD™ WORKS[^]

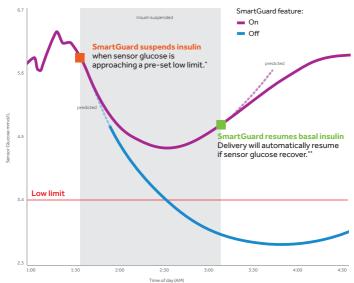

^SmartGuard Technology is activated when the MiniMed 640G is paired with continuous glucose monitoring *The dynamic suspend feature is based on certain criteria: sensor glucose must be within 3.9mmol/L of the low limit and predicted to be 1.1mmol/L above the low limit within 30 minutes AND the pump must not be in the refractory period. ** The dynamic resume feature is based on certain preset criteria: sensor glucose must be 1.1mmol/L above the preset low limit and predicted to be 2.2mmol/L above within 30 minutes AND insulin must have been suspended for at least 30 minutes.

INSULIN DELIVERY FOR EVERY BODY



The infusion set consists of tubing that carries insulin from the pump to you.

On one end of the tubing is the reservoir connector that is attached to the reservoir which holds the insulin. On the other end is the insertion site section that attaches to you.

Make sure to change your infusion sets every 2-3 days to ensure continuous insulin delivery.

Medtronic offers four (4) different types of infusion sets for you to choose from:

Sure-T® infusion set

A simple steel needle infusion set with 90° insertion which is suitable for those who have had reactions to plastic cannula or a history of bent cannula. Also suitable for pregnant women (up to the 2nd trimester and small children.





Quick-set[™]infusion set

Our most popular plastic cannula infusion set with 90° insertion. Suitable for those who are less active, have limited dexterity and for alternative site usage.

Silhouette[™] infusion set

A flexible insertion angle plastic cannula infusion set with 30-45° insertion. Suitable for those who are active, lean or muscular. Also suitable for pregnant women (up to the 2nd trimester).





mio®infusion set

A convenient 'all in one' plastic cannula infusion set with 90° insertion. Suitable for those who are less active, and for alternative site usage.

GET THE FULL PICTURE WITH CARELINK




Our three-step approach to diabetes management is fully integrated. Your Medtronic pump will wirelessly connect to your CONTOUR® NEXT LINK 2.4 blood glucose meter, meaning all data is stored conveniently on your pump.



Our Carelink® Personal Software helps you visualise your diabetes information with easy to read charts, statistics and events that can help you identify and understand patterns and trends. With one simple upload, you and your healthcare team can make adjustments to your therapy or remotely.

Visit: carelink.medtronic.eu for more information

Example Carelink Report

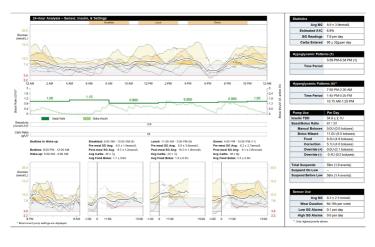

REAL STORIES FROM REAL PEOPLE[‡]



A passionate road cyclist, Jason is determined to continue to enjoy the freedom to exercise. It was a trip to an information evening that opened Jason's eyes to the possibilities of using a pump - something that he's been doing since 2010.

"When I was diagnosed with diabetes, both my brother and I were using MDI. I was the first to go onto an insulin pump and very quickly after that my brother saw the benefit and then he too went onto an insulin pump"

"Having a pump really changed my social life, especially when it came to going out for dinner."

"Smartquard gives me peace of mind!"

Check out more patient stories by visiting: medtronic-diabetes.com.au



By choosing a MiniMed[™] 640G Insulin Pump you have joined the global Medtronic Diabetes Family.

For a full range of accessories to style your pump visit: eshop.medtronic-diabetes.com.au

REFERENCES

- 1. J. C. Pickup and A. J. Sutton Severe hypoglycaemia and glycemic control in Insulin Dependant Diabetes: meta-analysis of multiple daily insulin injections compared with continuous subcutaneous insulin infusion Diabetic Medicine 2008:25, 765–774.
- 2. Bergenstal RM1, Tamborlane WV, Ahmann A, Buse JB, Dailey G, Davis SN, Joyce C, Perkins BA, Welsh JB, Willi SM, Wood MA; STAR 3 Study Group. Sensor-augmented pump therapy for A1C reduction (STAR 3) study: results from the 6-month continuation phase. Diabetes Care. 2011 Nov;34(11):2403-5. doi: 10.2337/dc11-1248. Epub 2011 Sep 20.
- 3. Assumes four injections per day for 30 days and one infusion set change every three days.
- 4. The Diabetes Control and Complications Trial Research Group. The effect of intensive treatment of diabetes on the development and progression of long-term complications in insulin dependent diabetes mellitus. N Engl J Med. 1993;329:977–986.
- 5. Raskin, P. et al. Diabetes Care 2003 Sep; 26(9): 2598-2603
- 6. Steineck, I et al. BMJ 2015; 350:h3234 2015 Insulin pump therapy, multiple daily injections, and cardiovascular mortality in 18168 people with type 1 diabetes: observational study.

 $\label{eq:minimed} \begin{tabular}{ll} MiniMed, Guardian \& CareLink are trademarks of Medtronic MiniMed, Inc. \\ @ 2017 Medtronic MiniMed, Inc. All Rights Reserved. \\ \end{tabular}$

‡ This patient testimonial relates to an account of an individual's response to treatment. The account is genuine, typical and documented. The response other persons have to treatment could be different. Please consult with you healthcare professional for information on whether the treatment is appropriate for you.

Medtronic

Medtronic Diabetes Australia 2 Alma Road Macquarie Park, New South Wales, 2113 Australia

24 Hour Global Helpline 1800 777 808

medtronic-diabetes.com.au